



Bindi's STORY

SPECIES	Equine
BREED	Quarter Horse
AGE	Yearling
SEX	Filly
INJURY	Traumatic
TREATMENT	4 total applications of ReCruit™ Topical
RESULTS	Almost complete healing of wound at 88 days after initial application

ReCruit™ Topical POWDERED EXTRACELLULAR MATRIX

Clinical Evaluation | Case Study

The product was applied to what type of wound?

Barbed wire fence traumatic injury to the cannon bone area of the left rear limb. Initial wound was inflamed and warm to the touch, but relatively clean.

How was the wound prepared?

It was cleaned and margins of the wound were lightly debrided (mechanically).

How was the product applied?

Product was "puffed" onto the wound, applying a full coat of product across the wound bed. A half bottle (1/2 gram) was applied at the initial application. After applying the product, a non-adherent pad and bandage was applied to protect the wound.

Were bandage changes needed?

Yes - bandage changes occurred every 3-5 days for 4 weeks then weekly for 4 more weeks, which were placed to protect the wound from the outside environment.

How many applications of the product were needed?

4

Did the patient experience any complications due to the product?

No; however, 17 days after the initial application, Bindi re-injured herself.

How did the product perform?

Excellent - it enhanced healing time and improved patient comfort.



DAY 0



DAY 14



DAY 17



DAY 88

Re-injury



Learn more about how ReCruit™ Topical is helping
cookanimalhealth.com/recruit